



# Brain Waves Analyser

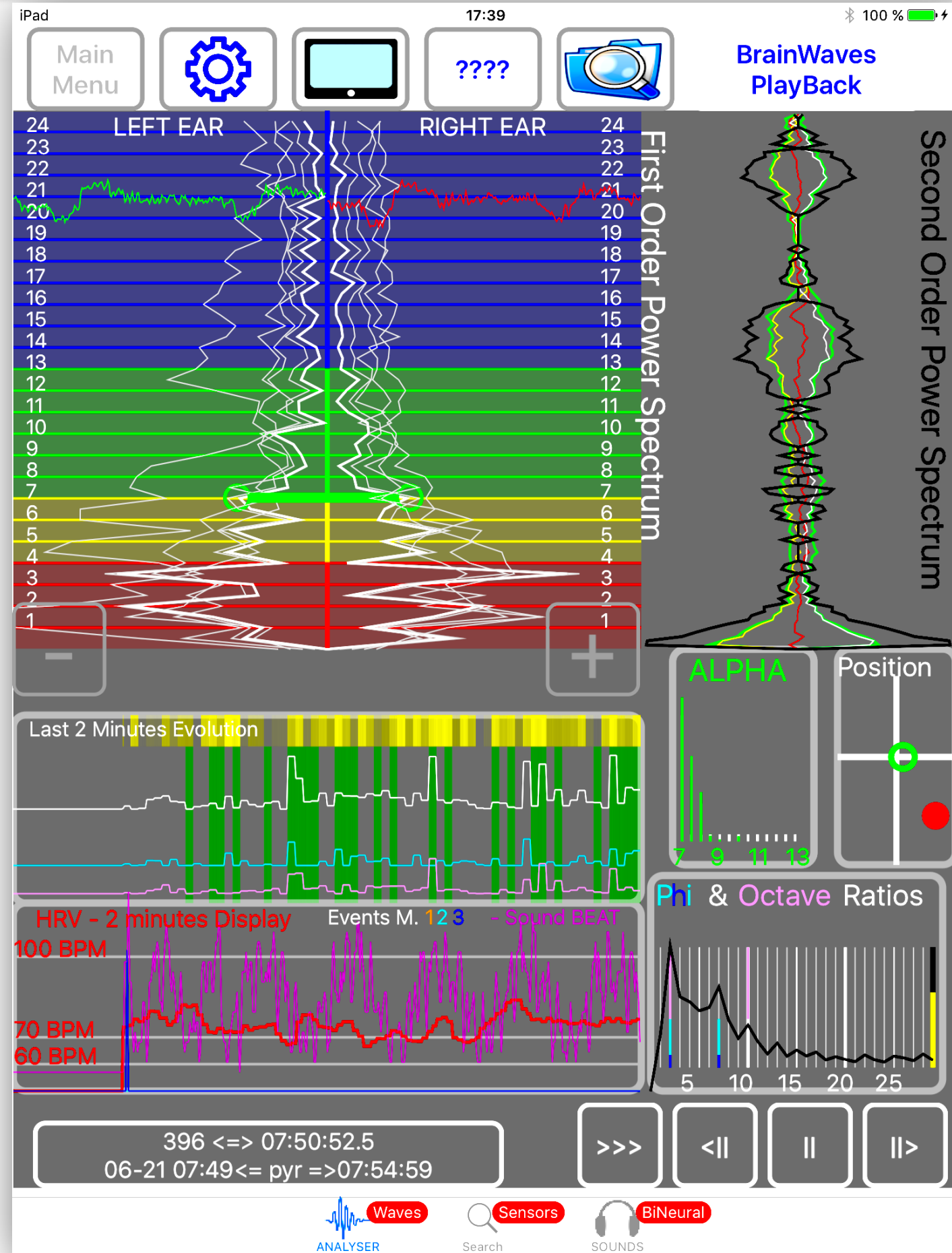

Files Manager  
(to PlayBack a recording)

File Name	File Size
2017-06-21 07:49 pyr	8.5 MB
2017-06-20 07:53 pyrsnd	10.4 MB
2017-06-19 07:52 tsst	2.2 MB
2017-06-19 07:51 tst	0.3 MB
2017-06-19 07:37 pyrsnd	9.7 MB
2017-06-18 07:57 pyr snds	11.2 MB
2017-06-16 08:50 TESTING	1.8 MB
2017-06-16 07:34 pyrsndpol	12.5 MB
2017-06-15 08:41 pyrsndlow	11.3 MB
2017-06-14 15:02 tryy	0.4 MB

# Files Manager

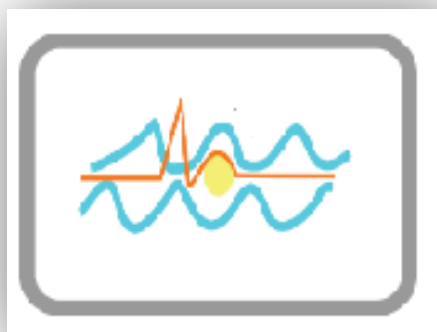

To go back to Analyser Screen

iPad 10:18 100%


BrainWaves  
PlayBack

	2017-06-21 07:49 pyr	8.5 MB
	2017-06-20 07:53 pyrsnd	10.4 MB
	2017-06-19 07:52 tstt	2.2 MB
	2017-06-19 07:51 tst	0.3 MB
	2017-06-19 07:37 pyrsnd	9.7 MB
	2017-06-18 07:57 pyr snds	11.2 MB
	2017-06-16 08:50 TESTING	1.8 MB
	2017-06-16 07:34 pyrsndpol	12.5 MB
	2017-06-15 08:41 pyrssndlow	11.3 MB
	2017-06-14 15:02 tryy	0.4 MB
	2017-06-14 08:44 danjune1	1.1 MB
	2017-06-14 08:02 BUG BUG	0.1 MB
	2017-06-14 07:51 pyrsndpol	11.1 MB
	2017-06-13 07:49 pyrsnd pll	15.8 MB

**Touch the file for more infos  
and/or  
PlayBack the selected file**

From left to right :

- Date
- Time
- Session Name
- Size of the file

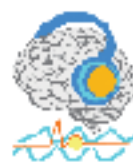

2017-06-14 08:44 danjune1

1.1 MB

6-14 07:51 pyrsndpol

6-13 07:49

6-12 17\_56

6-12 17\_45

6-12 16\_35

6-12 16\_27

6-12 16\_16

6-12 16\_11 agun

### File Selected

2017-06-12 17\_45 juan binaural

END: 2017-06-12 17:49:27

NOTE:  
deep meditation

[BACK to the LIST](#)

[PLAY](#)

### File selected:

- Start Time/Date
- End Time
- Comment to this recording (NOTE)


### SELECT :

**BACK to the LIST**  
to select another file

**PLAY**  
to playback the file in the Analyser


## To DELETE (remove) a file from the list :

SWIPE left the selected file, than touch the red DELETE button

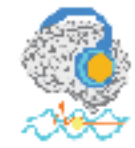

2017-06-19 07:37 pyrsnd

9.7 MB

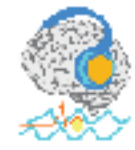

2017-06-18 07:57 pyr snds

11.2 MB

017-06-16 08:50 TESTING

1.8 MB

Delete



2017-06-16 07:34 pyrsndpol

12.5 MB



2017-06-15 08:41 pyrssndlow

11.3 MB



2017-06-14 15:02 tryy


0.4 MB

# Play Back buttons

A refresh of the display is done each 0.25 second of the recording.  
NEXT & PREVIOUS buttons will display next/previous 0.25 second.

And Analysis Frame Number is increased every 0.25 sec of the recording.

Analysis Frame  $\Leftrightarrow$  Running Time  
Starting time  $\Leftarrow$  Session name  $\Rightarrow$  End time

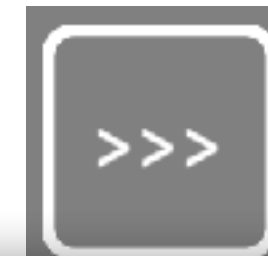

PLAY - PLAY Fast  
PAUSE  
Next  
Previous

45  $\Leftrightarrow$  08:51:07.8  
06-16 08:50 $\Leftarrow$  TESTING  $\Rightarrow$  08:52:25

