


**To connect a sensor**  
Touch the lower middle button

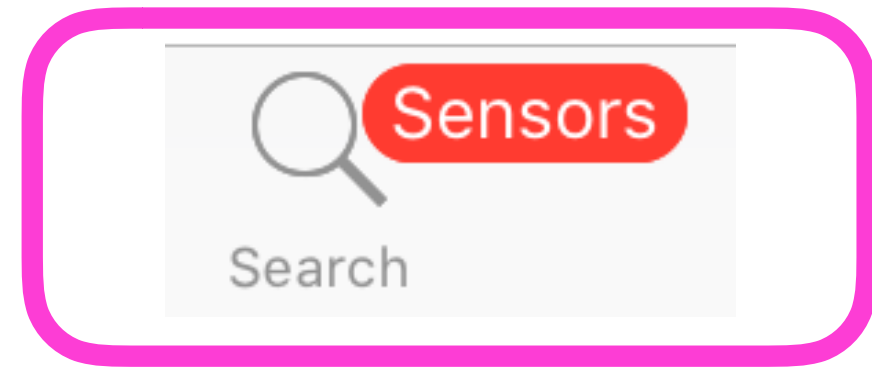



Sensors must be connected before playing a live session.  
Make sure BlueTooth is ON !!

Not necessary to connect the sensor if you want to play back a previous recording.

**3 main parts in the App:**


- Brain Waves Analyser
- Connection with Sensors
- Binaural Sound Generator

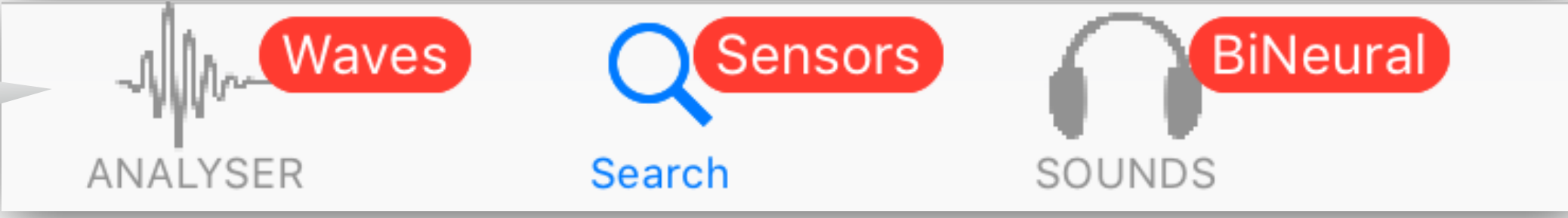

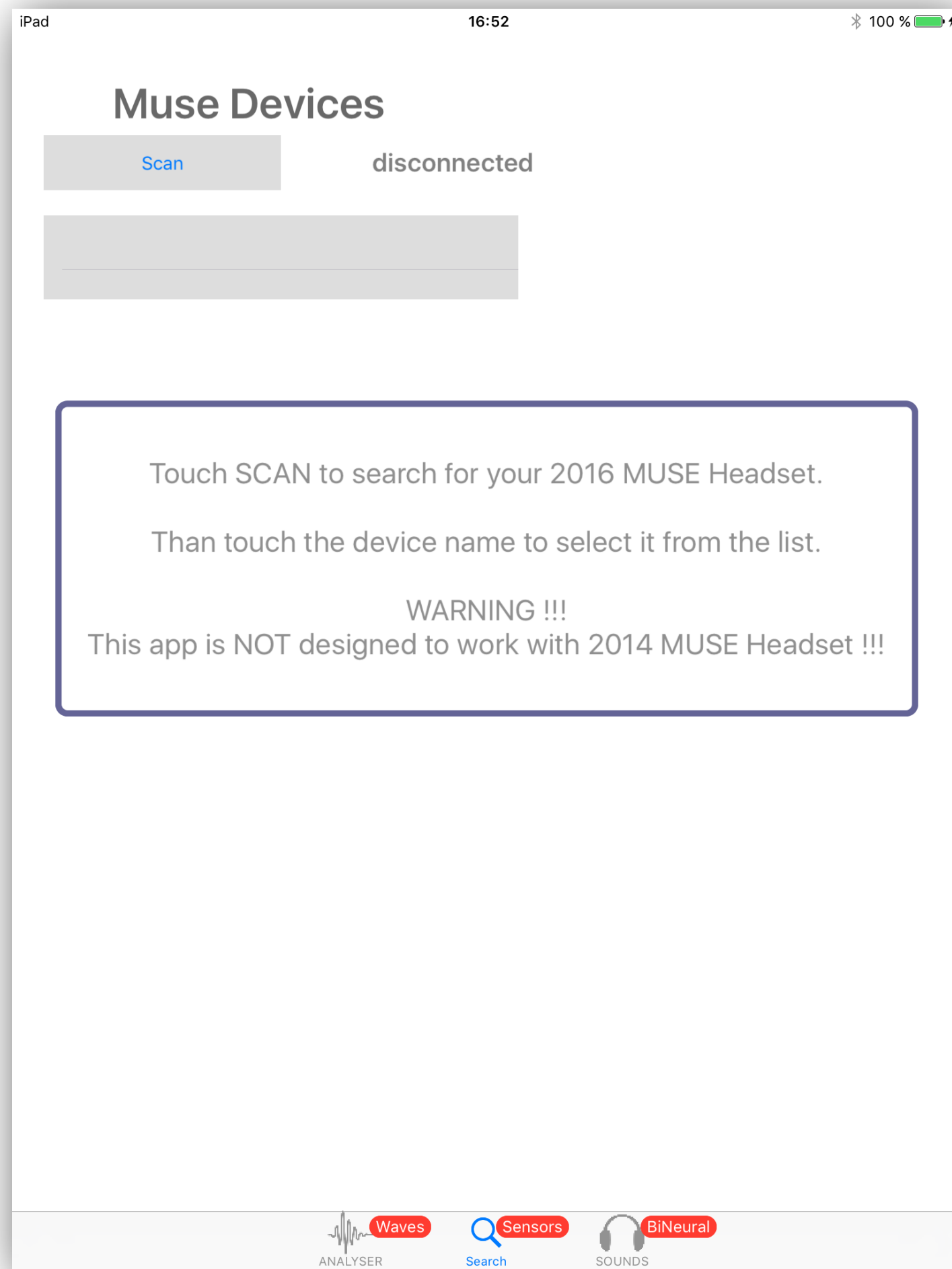


**!! WARNING !!**  
This App can only work with Muse 2016 headset.  
**!! NOT compatible with Muse 2014 headset !!**

Touch the SCAN button  
The name of your headset is displayed in the list.

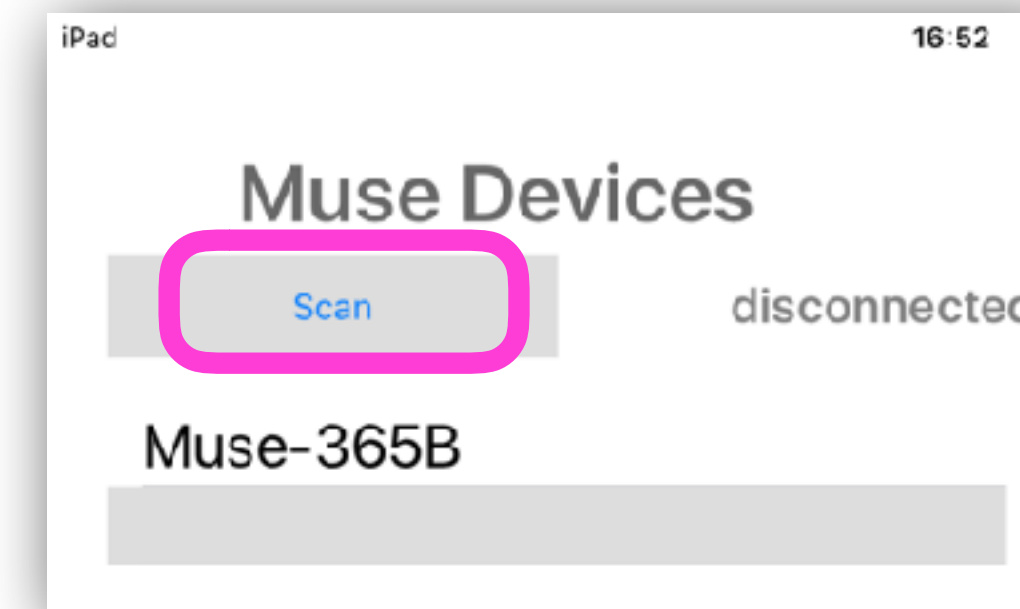

Touch the name in the list  
The serial number and the battery level are displayed



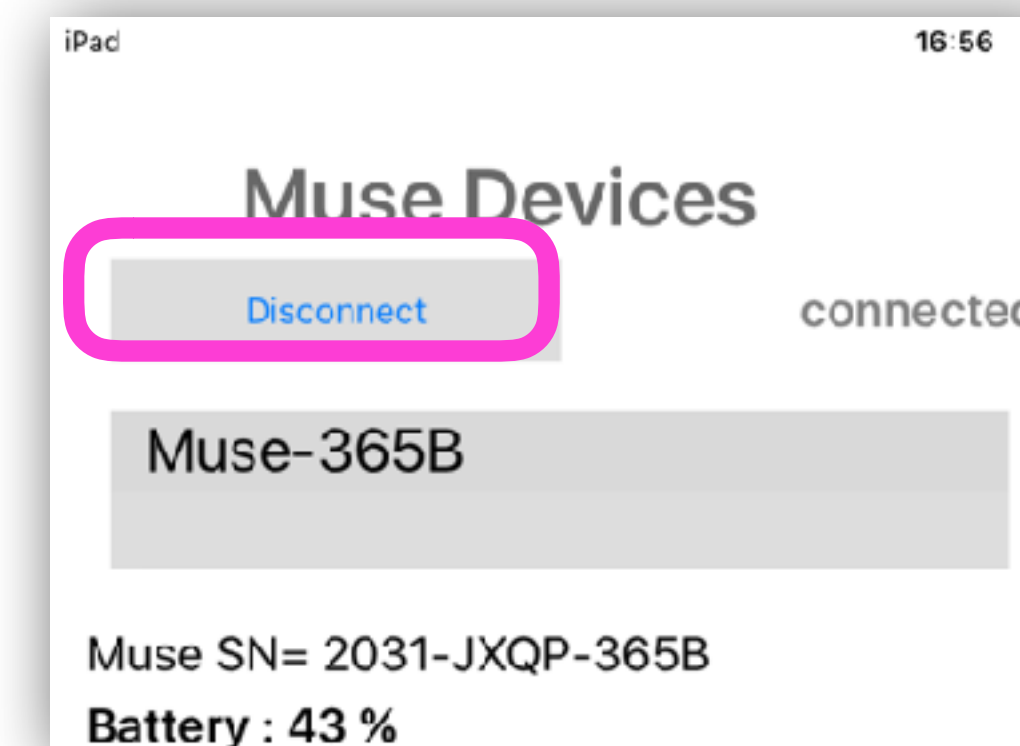


## For Brain Wave Sessions


Please check the headset is correctly fit.

A correct fit should give a noise lower than 10%

To get a better signal, wet the rubber bands of the headset related to the back of your ears.

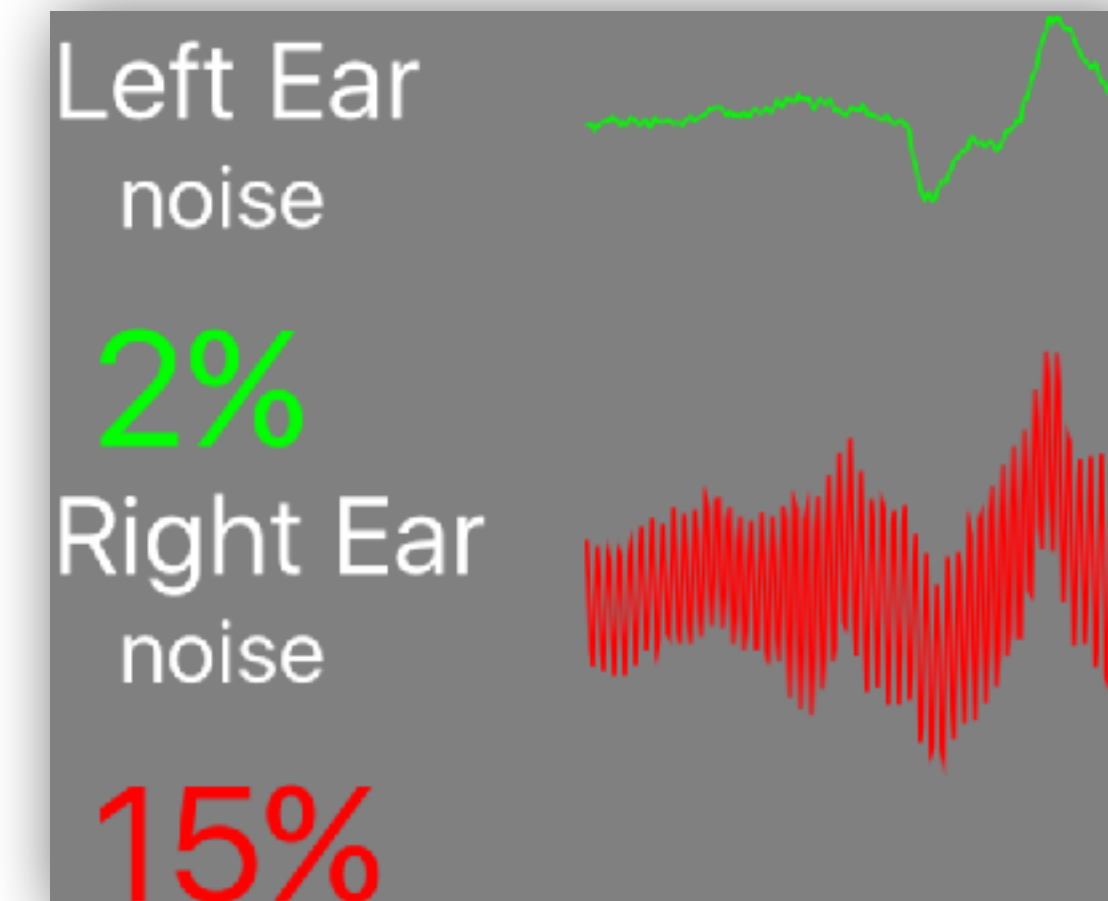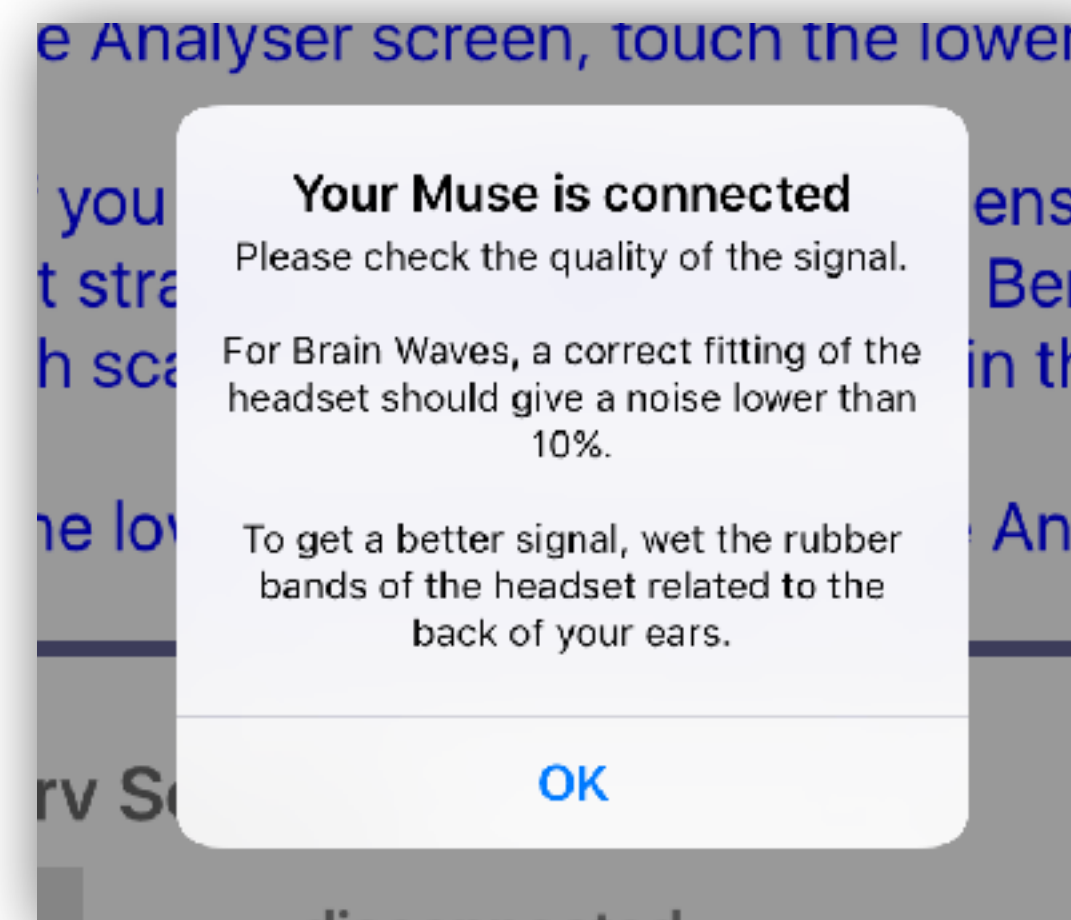
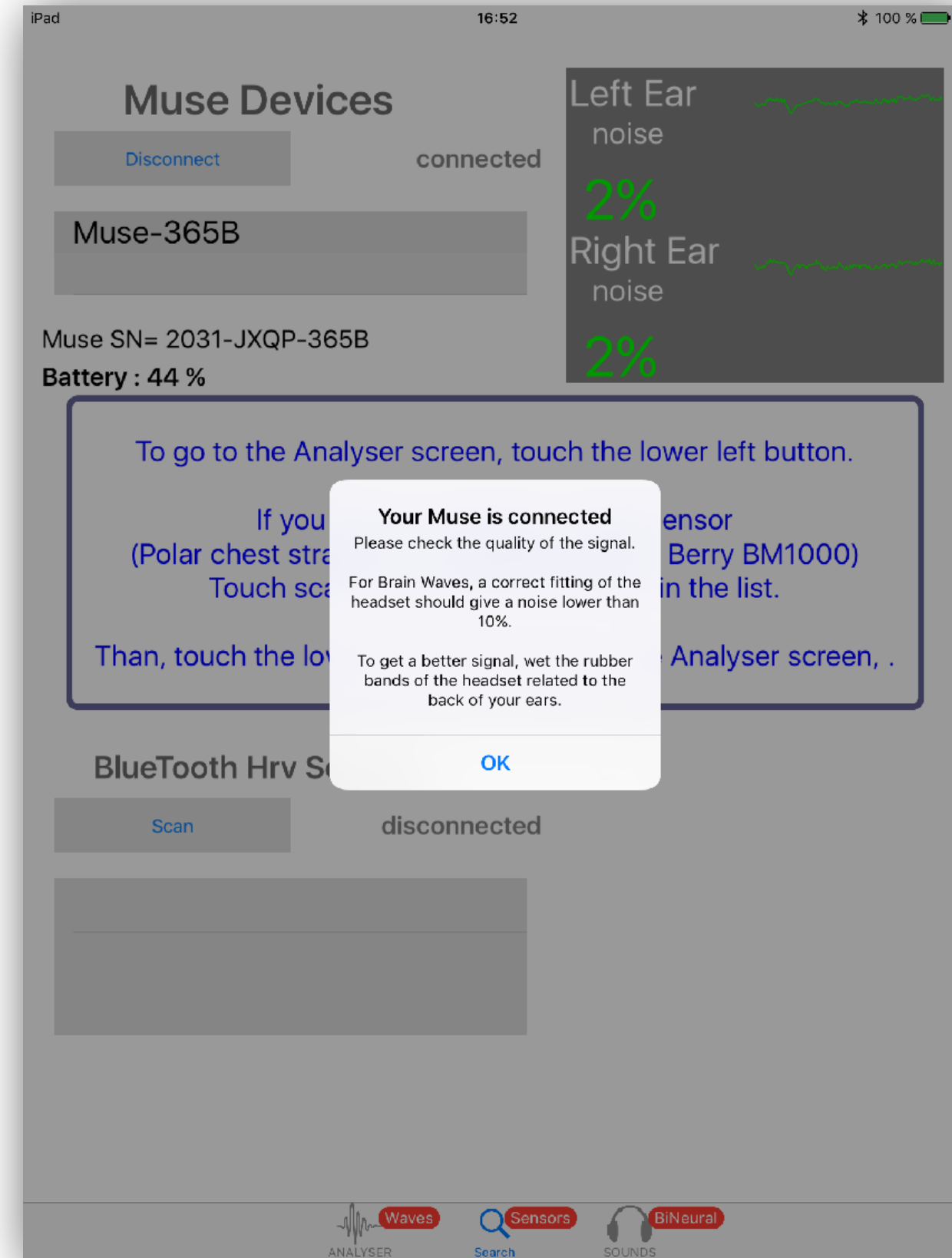


**You can add (optional)  
a Polar H7 Chest strap HRV heart rate sensor**  
enabeling you to see your heart rate/breath  
coherence curve compared to your brain waves.

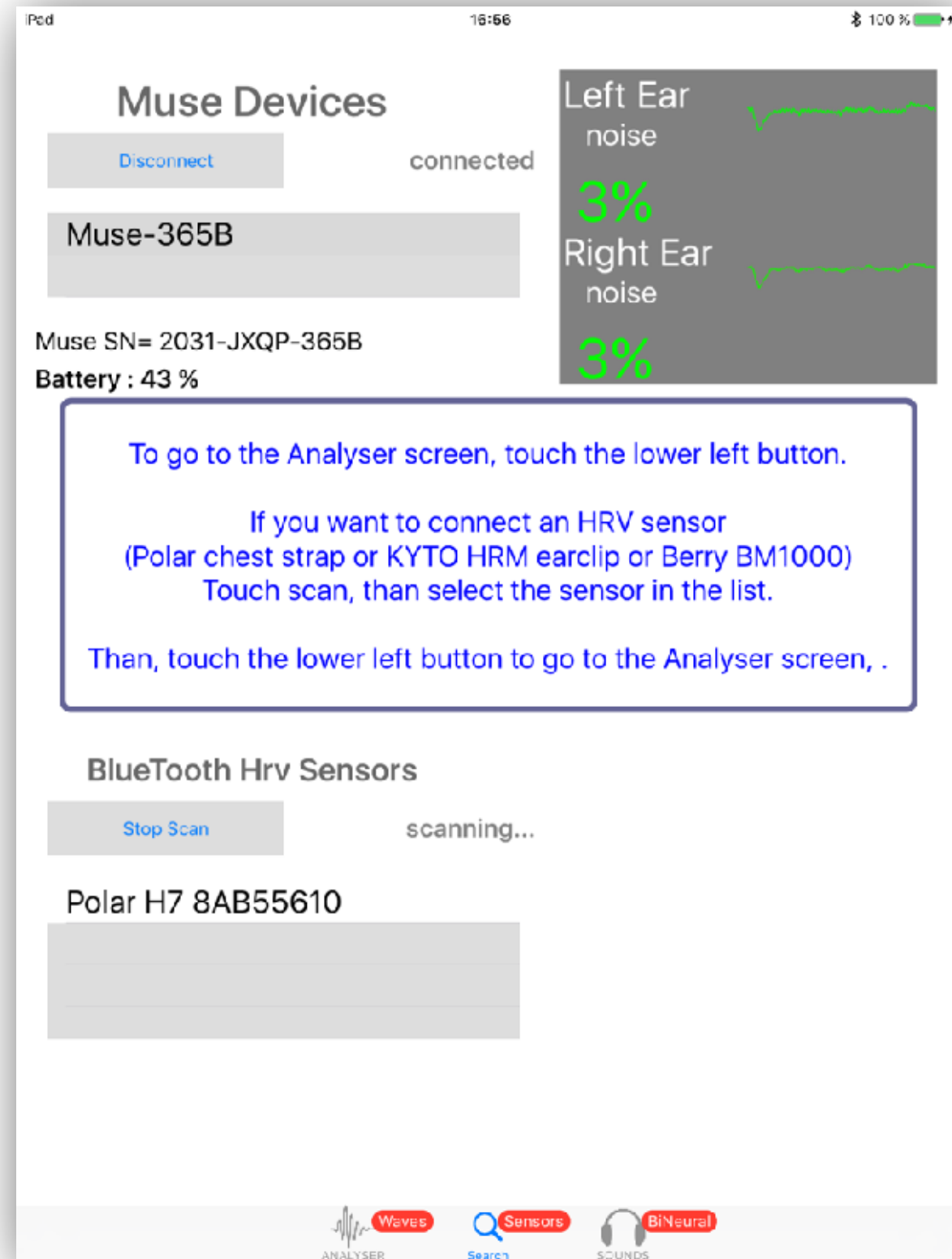


Touch the SCAN button


The name of your  
headset is displayed  
in the list.

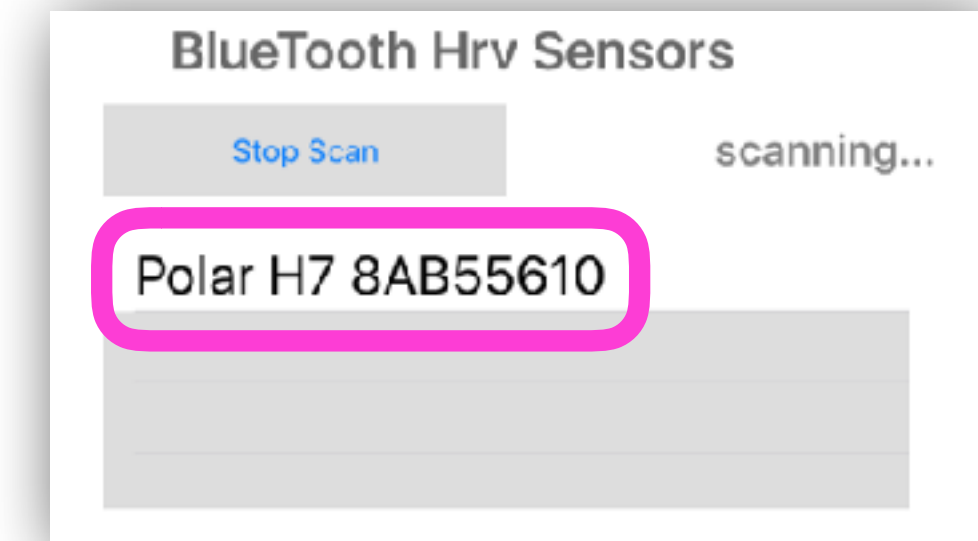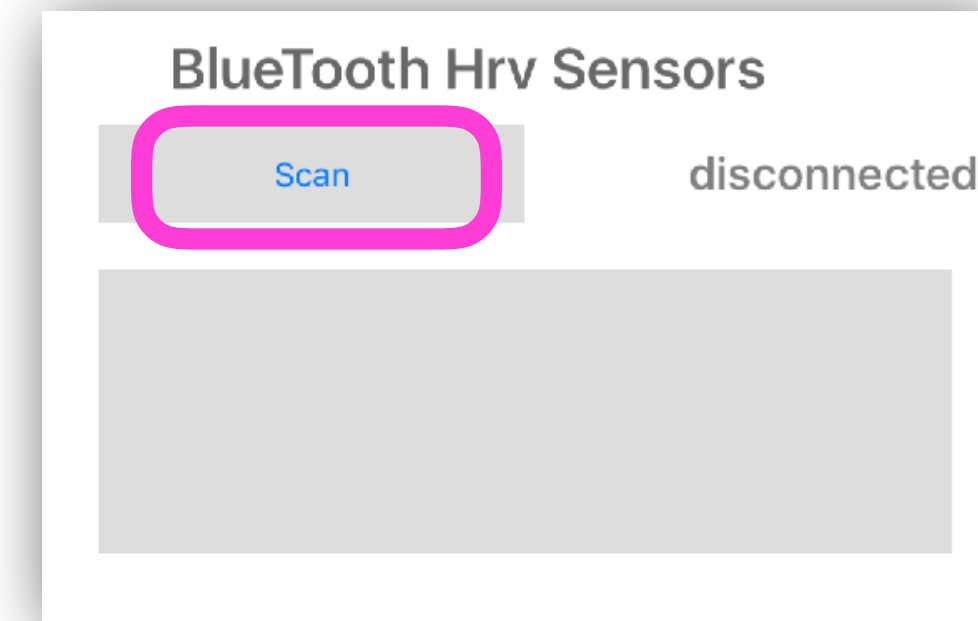

Touch the name  
in the list

The BPM  
are displayed

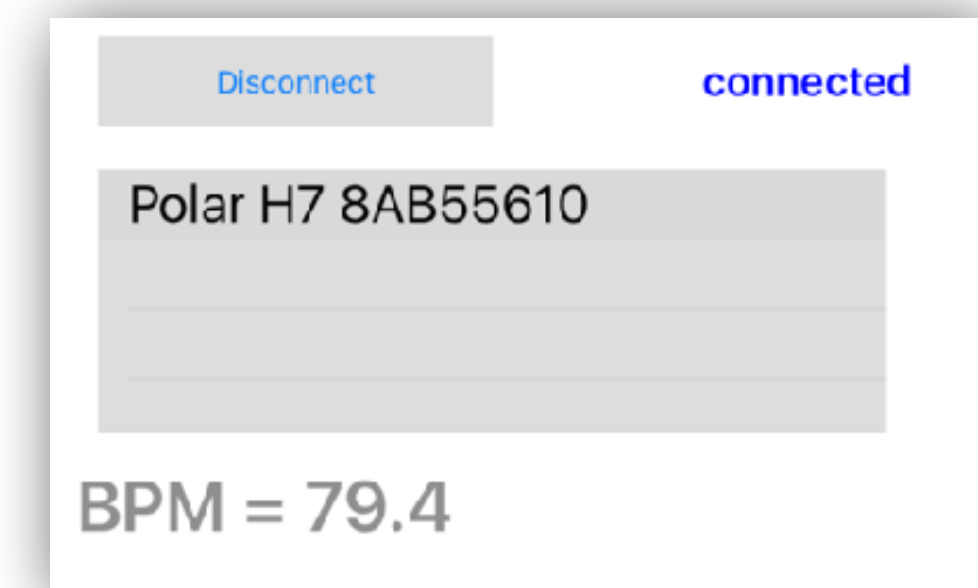

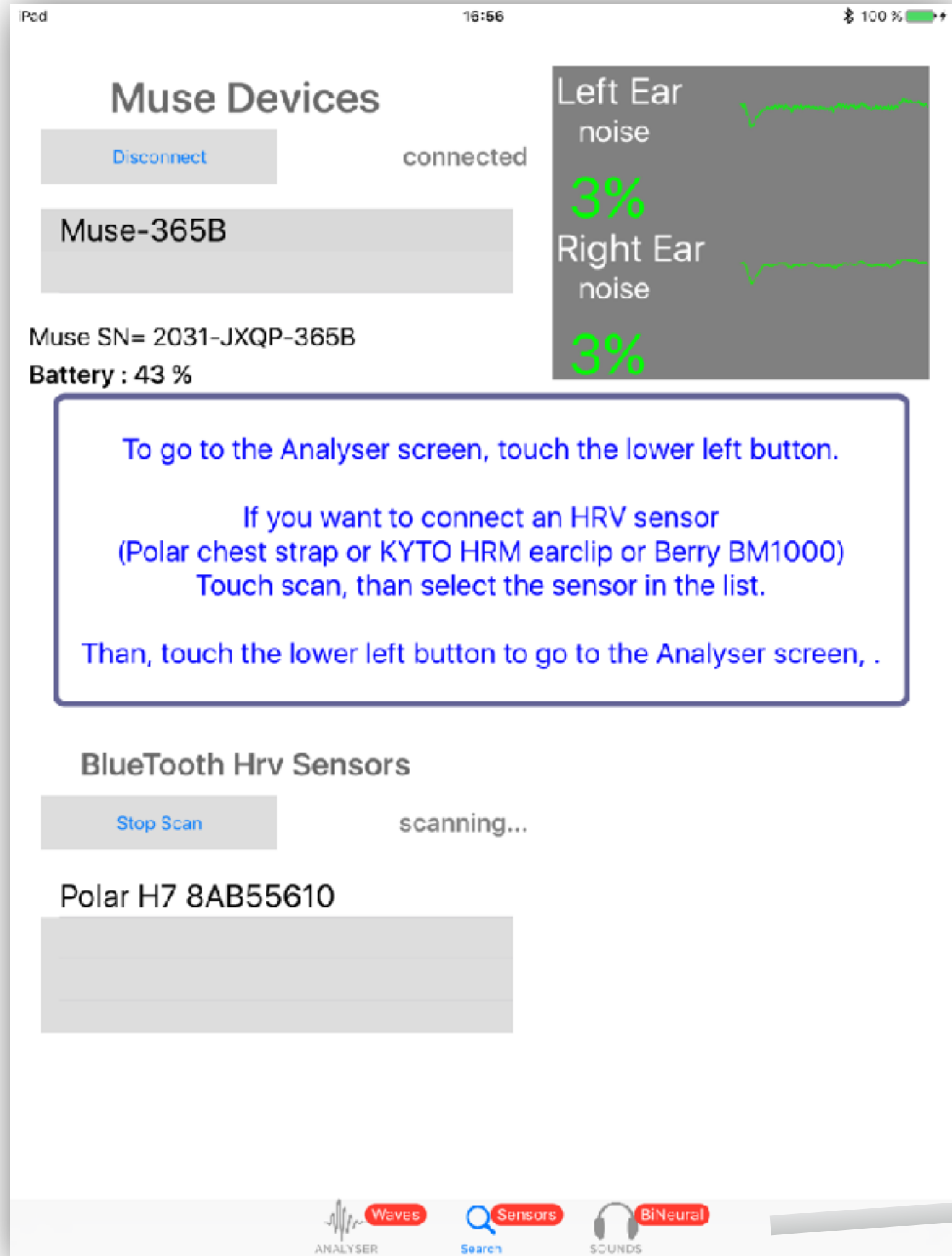

**To Go to the Waves analyser**  
Touch the lower left button

